nosiboo<sup>\*</sup>go<sup>®</sup>  
portable nasal aspirator

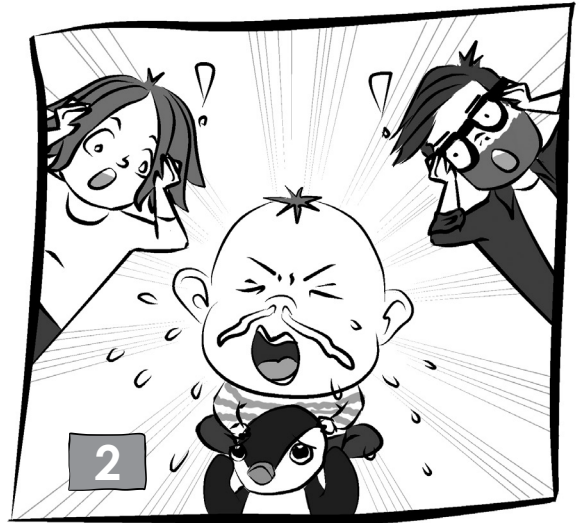
2023.04.01. Nosiboo\_Go\_QG\_Digit\_1.4

**nosiboo<sup>\*</sup>go<sup>®</sup>**  
portable nasal aspirator

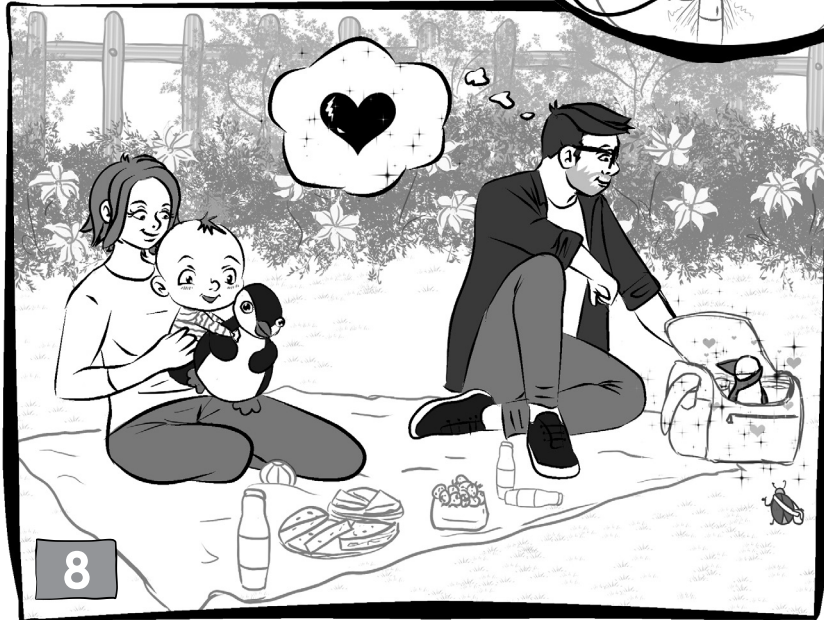
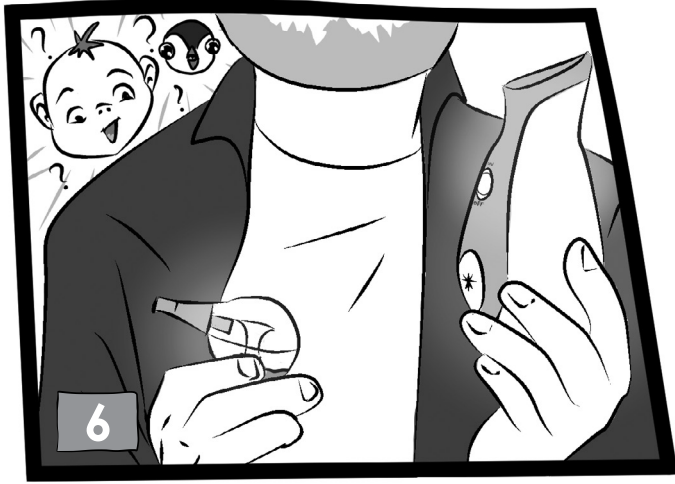
Before using the Nosiboo Go, carefully read this guide and the **User Manual** as well! Please keep this guide for future reference.

# Pingo & Nosiboo


# IN THE BOX

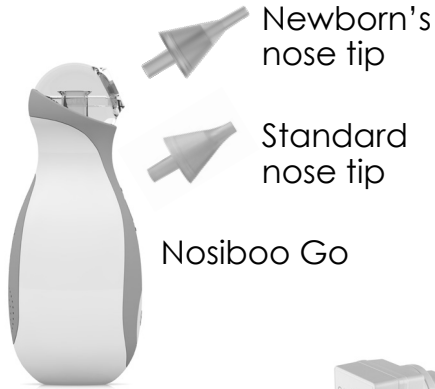Nosiboo Go

International charger with 3 power plugs

Brush

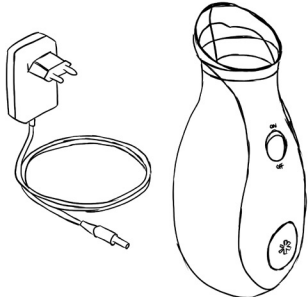
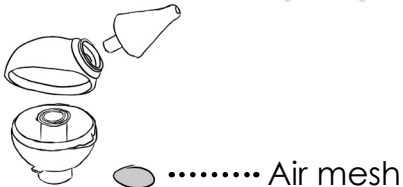
Sack



Quick start guide

Safety instructions

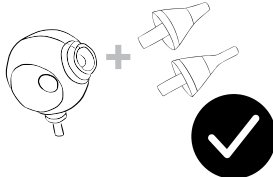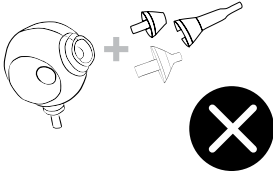
 **Use original Nosiboo parts!**  
**Our nasal aspirators work effectively only with original Nosiboo parts/accessories.**

Contact the distributor if any part is missing!

# USING THE NOSIBOO GO


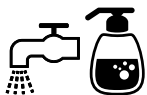
The device should be used for removing the nasal mucus from children's nose. It should not be used for any other purposes.


First clean\* and dry the parts of the Penguin head.

\*Check the "CLEANING" section below for more details about the proper cleaning method.

**Assemble the Penguin head.**

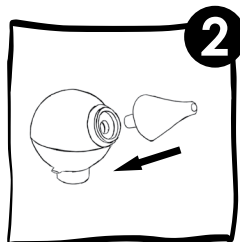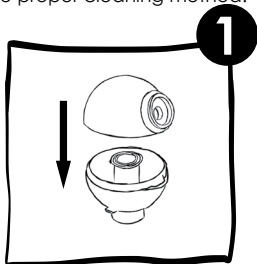
**Insert the nose tip into the Penguin head.**


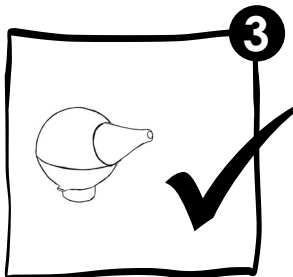
\* Newborn's nose tip:  
0-6 months

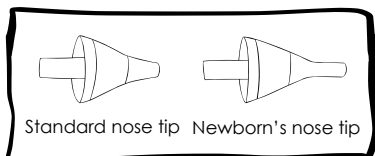\* Standard nose tip:  
6 months and older

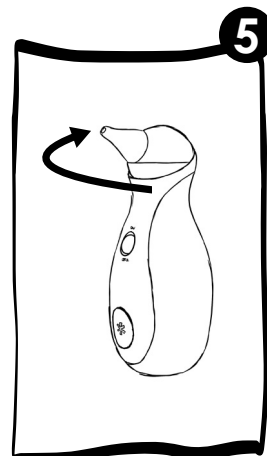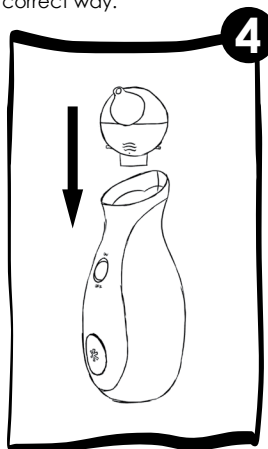
Make sure the air mesh is in its place, the Penguin head is assembled properly and installed on the device in the correct way.

**The device is ready to use.**

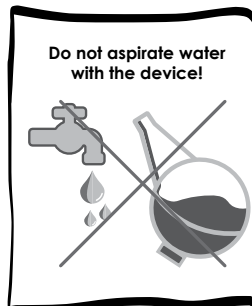**Aspirating water with the electric device will result in loss of warranty.**

**Do not aspirate water with the device!**

**Do not use the nasal aspirator for nasal washing!**

# INSTRUCTIONS FOR USE & SETTINGS

## 1 Getting started

Simply turn it on.

The effective and strong (max.) suction power has been proven safe for nasal aspiration by ENT doctors.

## Hold your child in a stable position.

- \* Aspirate the nose of infants after laying them into a stable position and hold their head during the process.
- \* You may also sit them in an infant bouncer or infant seat.
- \* If the child is already able to sit, sit them in your lap. In this case, do not lay the child on their back during the aspiration.

## 2

## 3 Insert the clean, dry nose tip into the child's nostril.

- \* Move the nose tip around gently to make sure that you get all areas of the nostril.

## 3

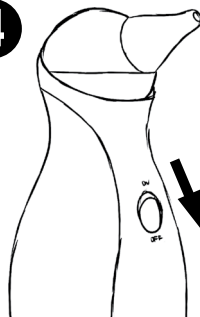
If you experience that mucus or liquid has entered the device then stop using the Nosiboo Go, turn it off and contact the Nosiboo customer service team right away.

- \* Don't turn the device upside down meanwhile use.

## 4


Switch off the device after use.

- \* We highly recommend removing the Penguin head and cleaning it right away.



## 5 Charging

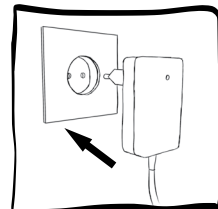- \* After use, or if the device's battery just ran low, wait a few minutes until the device cools down and then start charging it.
- \* Green LED - Fully charged
- \* Red LED - Still charging
- \* While being charged, the device cannot be used.
- \* Use the power plug suitable for the socket.


# CLEANING

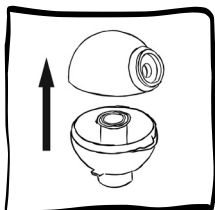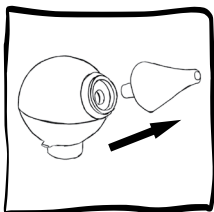Detach the Penguin head in the following order:

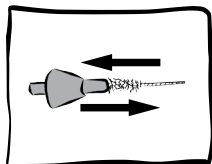\* Wash the parts with liquid soap and lukewarm water.


Use the brush to clean the nose tip.


\* You can soak the parts in a disinfection liquid, just make sure the liquid is safe to use for baby equipment.


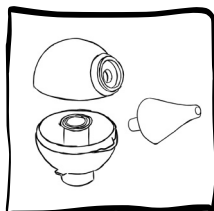
\* Let the parts dry completely before the next use.

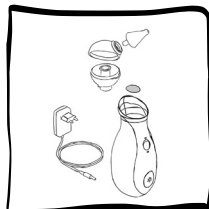\* It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer or baby bottle sterilizer for cleaning the spare parts because they might deform by heat!

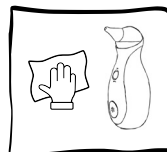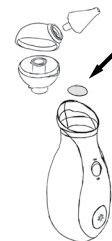- \* The air mesh is there to prevent solid objects and dust from entering the housing.
- \* The air mesh is not there to absorb any mucus or liquid.
- \* If the air mesh is dirty or clogged, replace it.
- \* It can function well for a very long time next to the proper usage of the device.

- \* Clean the housing with a dry, soft cloth.
- \* Do not use any liquids, detergents or chemicals.
- \* Keep the device in a place where it is protected from dust, direct sunlight and heat.

## HINTS & TIPS

- \* The Penguin head must be assembled and disassembled in the right order.
- \* Do not cover the vents on the back of the device while in use.
- \* Dry, very thick mucus can be dissolved by a mild saline solution. After using it, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.
- \* Do not use more than 5 ml of saline solution at once.

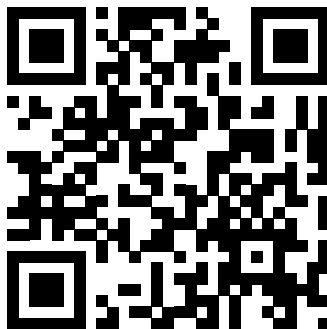
- \* The device should only be used by adults.
- \* Do not allow children to play with the device, it is NOT A TOY.

# User manual

#### AFTER-SALE SERVICE

For more information about usage, cleaning, and for troubleshooting, see the User Manual.

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